


REVISIONS			
REV	DESCRIPTION	DATE	APPROVED
A	RELEASED	12-8-00	

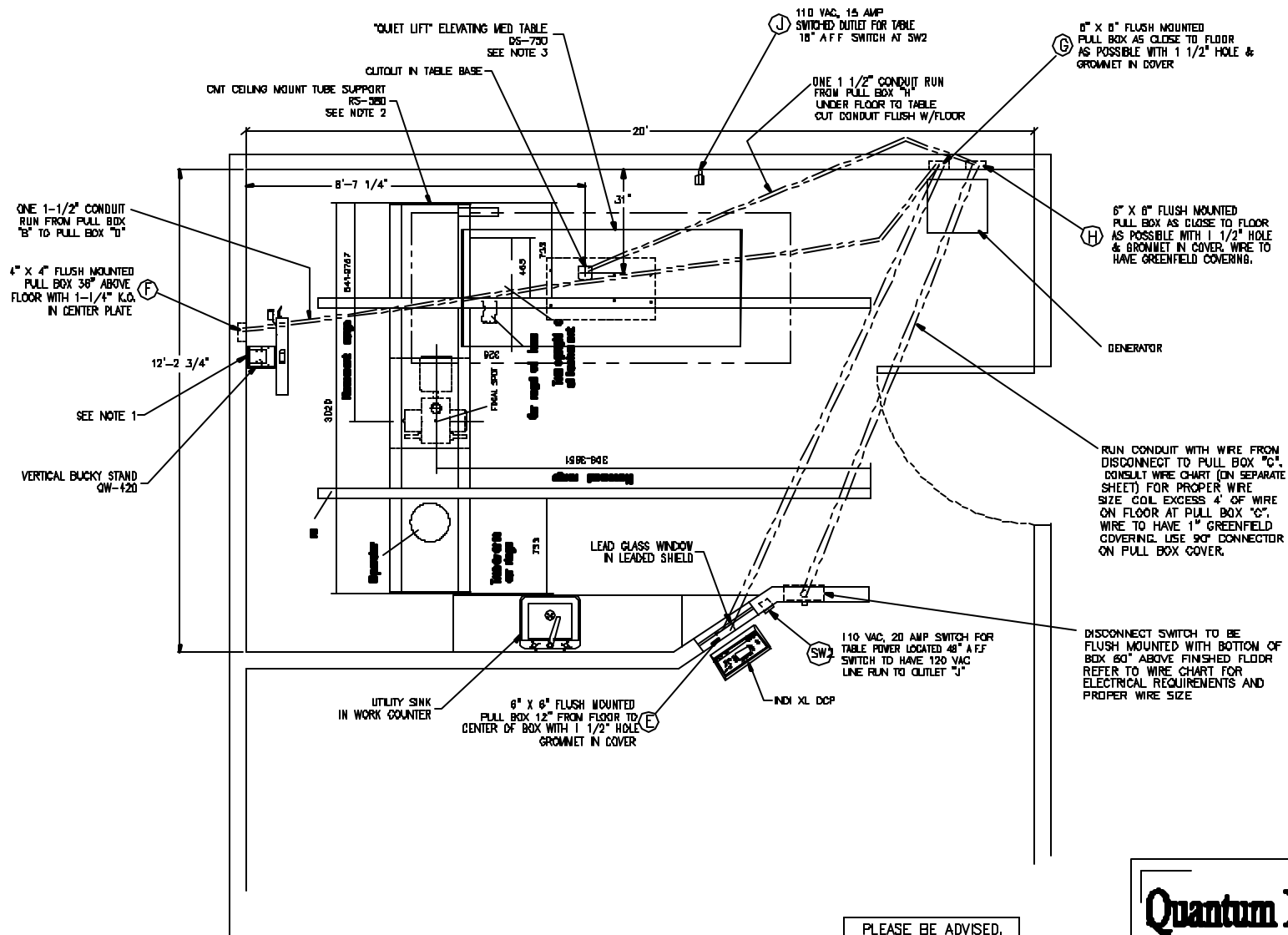

NOTES:

- BUCKY STAND MOUNTING POINTS MUST WITHSTAND APPROX. 100 LBS [45.36 kg] PULL.
- CEILING MUST BE CAPABLE OF SUPPORTING A 1000 LB [453.6 kg] DYNAMIC LOAD.
- QUIET-LIFT MOUNTING POINTS MUST WITHSTAND APPROX. 500 LBS [226.8 kg] PULL.

PLEASE BE ADVISED.

ALL ELECTRICAL CONDUITS, BOXES, WIRE, SWITCHES AND OUTLETS ARE NOT SUPPLIED BY QMI, LLC. QMI, LLC. ALSO DOES NOT SUPPLY, INSTALL OR PERFORM ANY PLUMBING OR CARPENTRY WORK AND/OR LEADING (INCLUDING LEADED WINDOWS).

Quantum Medical Imaging, LLC

EXAMPLE EQUIPMENT LAYOUT OF
CS-1 SYSTEM PKG

THIS LAYOUT MUST BE APPROVED BEFORE DRAWINGS CAN BE FINALIZED. THANK YOU.		DRAWN BY: DON WHITE	CHECKED BY:	DATE:	REVISION LETTER:
CUSTOMER:	DATE:	DATE: 12-8-00	APPROVED BY:	DATE:	A
REPRESENTATIVE:	DATE:	SCALE 1/4"=1'-0"			